


## Charging the H200 Wireless System

Connect the System Charger Set to the H200 Wireless Orthosis and the H200 Wireless Control Unit simultaneously. Charge the H200 Wireless System daily.

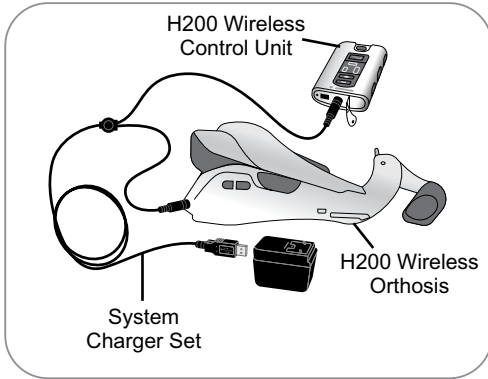

### Control Unit Charging Displays





	Flashes Yellow	Low Battery
	Rotating Green Circle	Charging
	Horizontal Green Line	Fully Charged

### Orthosis Charging Displays

	Status Light Flashes Yellow	Low Battery
	Status Light Flashes Yellow and Green	Charging
	Status Light Steady Green	Fully Charged

## Putting on the H200 Wireless Orthosis


- 1 Wet and attach the cloth electrodes.
- 2 Position the Orthosis on a stable surface, and put your hand into the spiral end. The Thenar (with attached electrode) should rest at the base of your thumb.
- 3 Position the wrist bridge comfortably on the back of your wrist.
- 4 Place your forearm in the Orthosis flexor support.
- 5 Close the extensor wing.
- 6 Fasten the wrist strap, if appropriate.

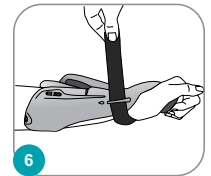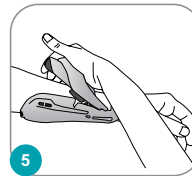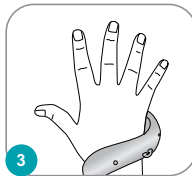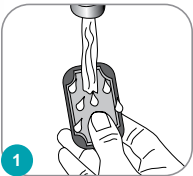







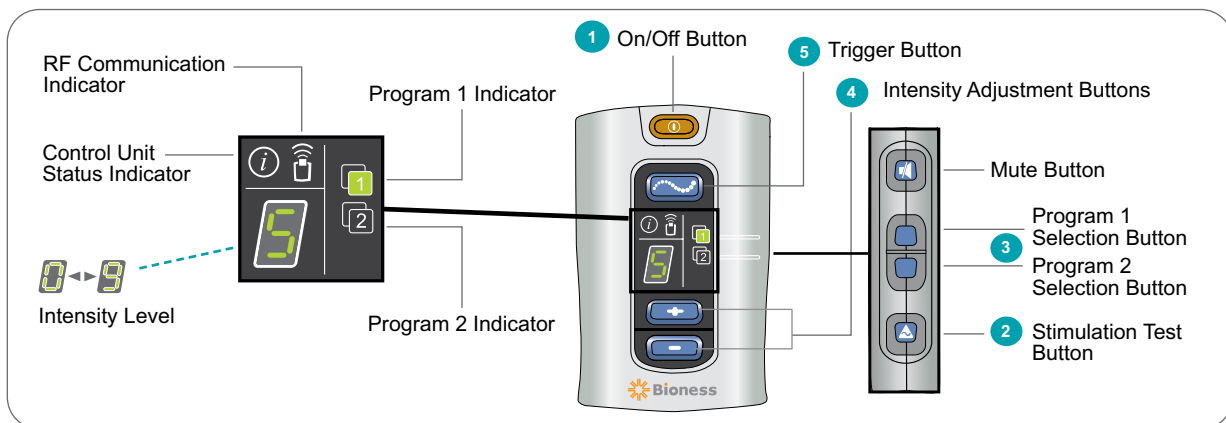

## H200 Wireless Orthosis Displays

System On Status Light Flashes Green	Stimulation On Stim Light Flashes Yellow	Stimulation Paused Stim Light Steady Yellow	Faulty Electrode Contact Status Light Flashes Red	Hardware/Software Error; Charging Error Status Light Steady Red_








# Operating the H200 Wireless System



- 1 **Turn on the system.** Press the  on/off button.
- 2 **Test the position of the Orthosis.** Press and hold the  stimulation test button once to test the extensor muscles. Release the button and press and hold again to test the flexor muscles. Repeat as needed.
- 3 **Select Program 1 or Program 2.** Press the Program 1 or Program 2 selection button on the side of the Control Unit. The selected program indicator will be GREEN:  or .
- 4 **If necessary, adjust the stimulation intensity level.** Press the  plus or  minus button to increase/decrease stimulation intensity in the Orthosis.
- 5 **Start a program.** Press the  trigger button on the Control Unit or the trigger button on the Orthosis to turn on stimulation. **To pause/resume a program,** press the trigger button on the Control Unit or the trigger button on the Orthosis.

**Note: Neuroprosthesis Programs cannot be paused.** Pressing the trigger button when stimulation is on initiates the next phase of the program. **To turn off stimulation,** press the  on/off button.

## H200 Wireless Control Unit Displays

	On/Off Button Flashes Green	System is On
	Trigger Button Flashes Yellow Rapidly	Stimulation is On
	Trigger Button is a Steady Yellow	Stimulation is Paused
	Program 1 Indicator is Green	Program 1 Selected
	Program 2 Indicator is Green	Program 2 Selected
	Control Unit Status Indicator is a Steady Red	Control Unit Malfunction: Charging Error; Registration Error; Software Error
	RF Communication Indicator Flashes Red	RF Communication Error

### Rx Only (US only)

For a complete list of all contraindications, warnings, adverse reactions, precautions, and care instructions please review the User's Guide.

©2023 Bioness Inc.

H200® Wireless, Bioness, the Bioness Logo® and LiveOn® are trademarks of Bioness Inc. in the United States or other countries. | [www.bioness.com](http://www.bioness.com)