

ATTENTION:

Read instructions carefully and completely before first use.

For full instructions on how to use the device, refer to the Instructions For Use booklet, supplied with the Controller.

IMPORTANT INFORMATION


⚠ WARNING

- **DO NOT** connect the Mouthpiece to the Controller before seeing your PoNS Trainer, to avoid premature expiration of the Mouthpiece.
- **DO NOT** place the Mouthpiece in your mouth while the Controller is charging or connected to your PoNS Trainer's computer.
- **DO NOT** wet the connector on the end of the Mouthpiece cord when cleaning the Mouthpiece. Always check and ensure the connector is dry before connecting and using the Controller.
- **DO NOT** use the PoNS if you are sensitive to nickel, gold or copper. The PoNS Mouthpiece contains nickel, gold and copper.
- **DO NOT** use your PoNS if you have open mouth sores, ulcers or tongue piercings.
- **DO NOT** eat, drink, smoke, or talk while using your PoNS, to avoid choking.
- If the materials on the Mouthpiece cause any allergic reactions or undesirable effects, discontinue using the PoNS immediately and contact your PoNS Trainer.
- **DO NOT** bite down on the Mouthpiece with excessive force.
- **DO NOT** use a cracked or broken Mouthpiece, to avoid choking.
- **DO NOT** connect the Mouthpiece to any other device or modify any part of the equipment.

Precautions

- Mouthpieces expire after 14 weeks of use. A Mouthpiece expiration message will show daily when the Mouthpiece has 21 days or less remaining. Plan for appropriate steps with your PoNS Trainer before your Mouthpiece expires.
- **The PoNS is for use by a single individual and must not be shared.**
- You must clean your Mouthpiece before first use. See 'Care and Maintenance' instructions.

WHAT'S INCLUDED

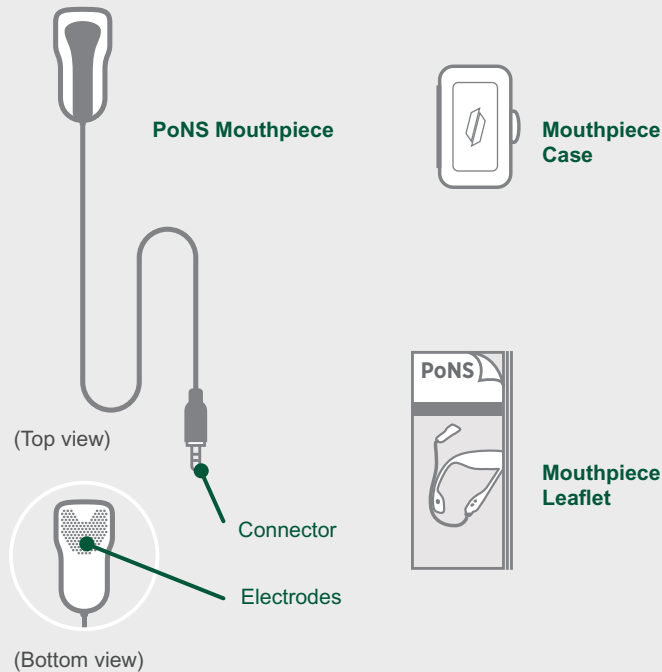

CARE AND MAINTENANCE

Cleaning your PoNS Mouthpiece

Follow these steps to clean the Mouthpiece before the first use as well as after each use and before storing.

⚠ WARNING


DO NOT wet the Mouthpiece connector.

Always check to ensure the connector is dry before reconnecting to the Controller.

The Mouthpiece is for use by a single individual and must not be shared.

1. Brush and Rinse Mouthpiece

Gently brush the Mouthpiece with warm water and a soft toothbrush being sure to keep the Mouthpiece connector dry. Do not use toothpaste.

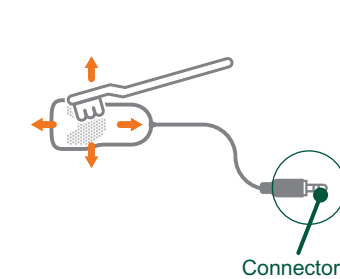


USING THE PoNS DEVICE

Clean the Mouthpiece before the first use and after each use. Check that the Controller is not damaged, the battery is charged and the Mouthpiece has not expired.

Note: A Mouthpiece can be stored for up to 4 years from the Manufacturing date (as printed on the label) prior to first use.

1. Attach Mouthpiece

Plug the Mouthpiece connector fully into the connection port on the Controller.

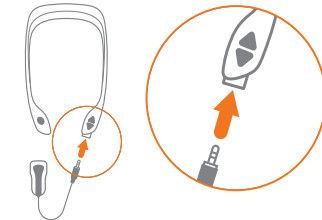

2. Turn On Controller

Press and hold ON/OFF button.




Your PoNS is ready for use when 'READY' appears on Screen.

Note: Other messages may appear. For full Controller operation instructions refer to the Instructions For Use booklet supplied with the Controller.

3. Place Controller

Place the Controller around the back of your neck.

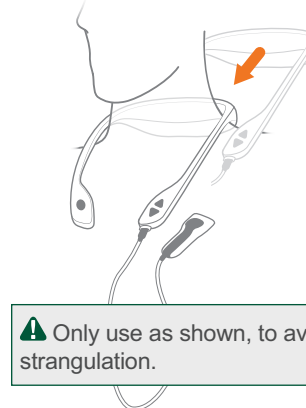

⚠ Only use as shown, to avoid strangulation.

Tongue Placement

The tip of your tongue should be just beyond the electrodes.

4. Insert Mouthpiece


Place the Mouthpiece in your mouth with the electrode side on your tongue.

Gently close your lips around the Mouthpiece and press your tongue towards the roof of your mouth.

⚠ **DO NOT** bite down on the Mouthpiece with excessive force.

5. Initiate Session

Press **SESSION** button. A full session lasts 20 minutes.

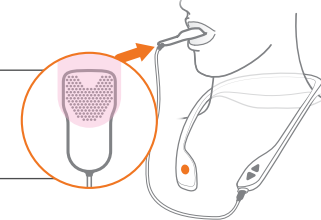

STORAGE

Once the Mouthpiece is dry, store it in the Mouthpiece case.



Note:

- *During each cleaning, check for worn or loose electrodes. If found, do not use and contact Customer Support.*
- *Discoloration of the electrodes or Mouthpiece can occur. This does not impact the use or effectiveness of the PoNS device.*

SYMBOLS AND MEANING

	Refer to Instructions for Use		Prescription only - device restricted to use by or on the order of a physician.
	Caution		Keep away from sunlight
	Classified as a Type BF applied part (electrically connected to the patient but not directly to the heart)		Temperature limit: minimum and maximum storage temperatures
	Please dispose product per WEEE (Waste from Electrical and Electronic Equipment) requirements		Mouthpiece Case recycling code - PP Polypropylene
	Do not wet the Controller or the Mouthpiece connector		Manufacturer
			Manufacturing Date